


Align IPR Composite

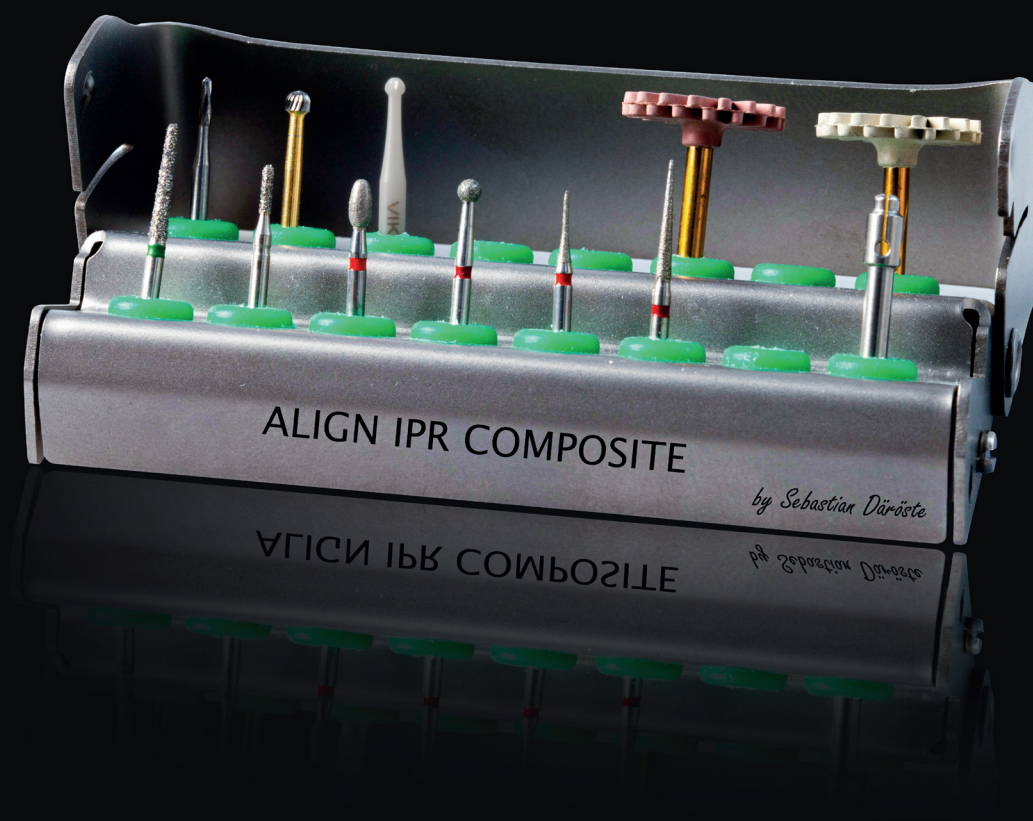

**"This kit is really everything I need
to shape and create beautiful
composite veneers and shape teeth
naturally while doing IPR!"**

**I love the quality of Viking burs, and
that they easily endure, being
used multiple times."**



Dr. Sebastian Däröste

Cosmetic Dentist and International
Speaker for Invisalign

500+ Invisalign
Cases

Invisalign Case
Café Mentor

Key Opinion
Leader for Invisalign

Member of European
Aligner Society

Postgraduate Diploma
in Clear Aligner Therapy

Affiliate Member of
Master Aligner Online Academy

Lecturer on Composite
Bonding & Aesthetic Dentistry

One of the World's youngest
Invisalign Diamond Providers

MSc in Specialist Practice of
Clear Aligner Orthodontics (ongoing)



SEBASTIAN DÄRÖSTE
AESTHETIC DENTISTRY



Veneer Prep - Coarse IPR.VP.C

Main veneer prep bur. Makes it easy to get a perfect finish line. Also good for composite veneers if a dark tooth needs prepping to give room for an opaque layer! Or for removing the bulk of old composite veneers.



Caries Prep Medium IPR.CP.M

Main cavity prep bur for your standard filling or to create a bevel.



Red Am. Football - Fine IPR.RF.F

For excess removal after fixed retainer placement. Also for occlusal adjusting or palatal anatomy after composite crowns.



Red Round - Fine IPR.RR.F

For occlusal adjusting or rounding off after a class 3 cavity prep.



Red Mosquito - Fine IPR.RM.F

Main IPR bur (0,3mm and above). For composite excess near margins or proximally.



Red Slice Bur - Fine IPR.RS.F

Heavy IPR (0,5mm) and reshaping long teeth after IPR. Great for primary & secondary anatomy after composite veneers.



Attachment Remover - Coarse IPR.AR.C

Main attachment removal bur. For GROSS REMOVAL- not all the way to enamel. Remove 80-90% with this and take the remaining with sof-lex.



Attachment Cleaner IPR.AC.F

Great for attachment excess clean-up. Also for removing lingual/palatal fixed retainer residue. Removes primarily composite, not enamel.



CERA 018 C.204.018

State of the art ceramic caries removal bur. Main caries removal bur after prep. Can be used many times. Does not rust during sterilization.



Fine Polisher 1 MAXS.F

First polishing. For composite veneers, temporaries, after attachment removal, fixed retainer placement.. After everything!



X-Fine Polisher 2 MAXS.XF

Second polishing. For composite veneers, temporaries, after attachment removal, fixed retainer placement.. After everything!



Pop-on Mandrel PoM.RA

For your Sof-Lex (Red, Orange, Yellow)

Experience the difference in bur technology

Taylor made solutions for your
needs on your terms.

Simple product identification
that you will remember.

Organized in an angled bur holder
for better view of burs chairside.

Room for some of your own favorites.

Compatible for both friction grip
and contra angle burs and polishers.

An optimal bur holder for rinse and
sterilizing cycles of burs and polishers.



Scan for products

